

Be READY VR-Nursing Spectator manual



System set

Wi-Fi router



PC or laptop




VR-Nursing Spectator minimal system requirements:

- CPU - not less than Intel Core i3 2.3 GHz or similar
- RAM - not less than 4 GB
- Video Card - NVIDIA GeForce GT 540M or similar
- Free disk space - not less than 3 GB
- Ethernet connector (or WiFi adapter)
- Monitor - not less than 1280x720 resolution
- Keyboard
- Computer mouse
- Windows 10 or 11 operating system

Updates


Update the VR-Nursing Mobile

- Open the application library in the main menu
- Choose the VR-Nursing – see details – more –update
- Wait for application to update

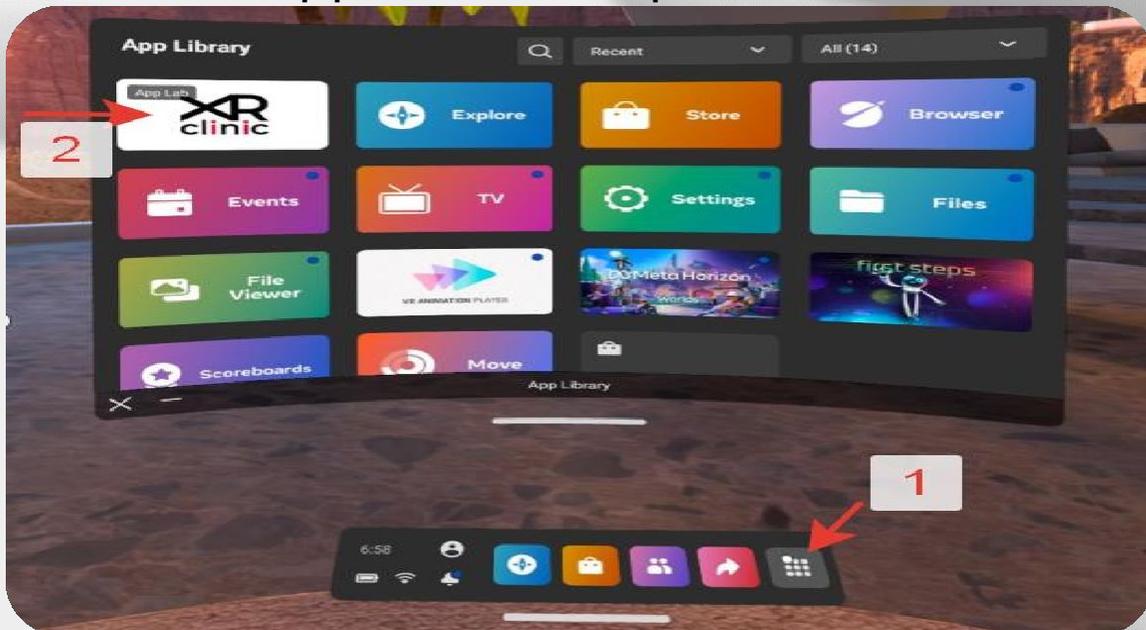


Update the VR-Nursing Spectator

- Go to: <https://mvredtech.com/manuals>
- Download the Spectator.
- Unzip the archive to any folder on your computer

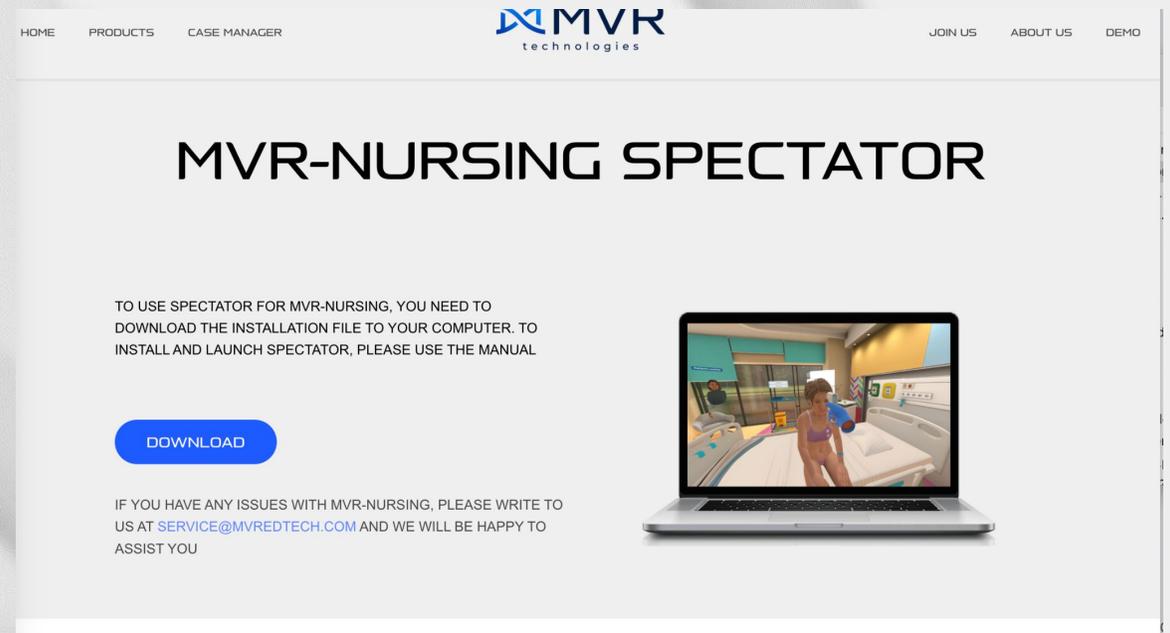


Spectator setup

- Connect your computer to the **same** Wi-Fi network as the VR headset
- Start “MVR Nursing” on the computer
- Launch the Be READY VR-Nursing Mobile application on the headset -> enter your name -> press Start new Session button
- Click on the computer in the Server list the Session with your naming and press “Spectate” button for real-time monitoring

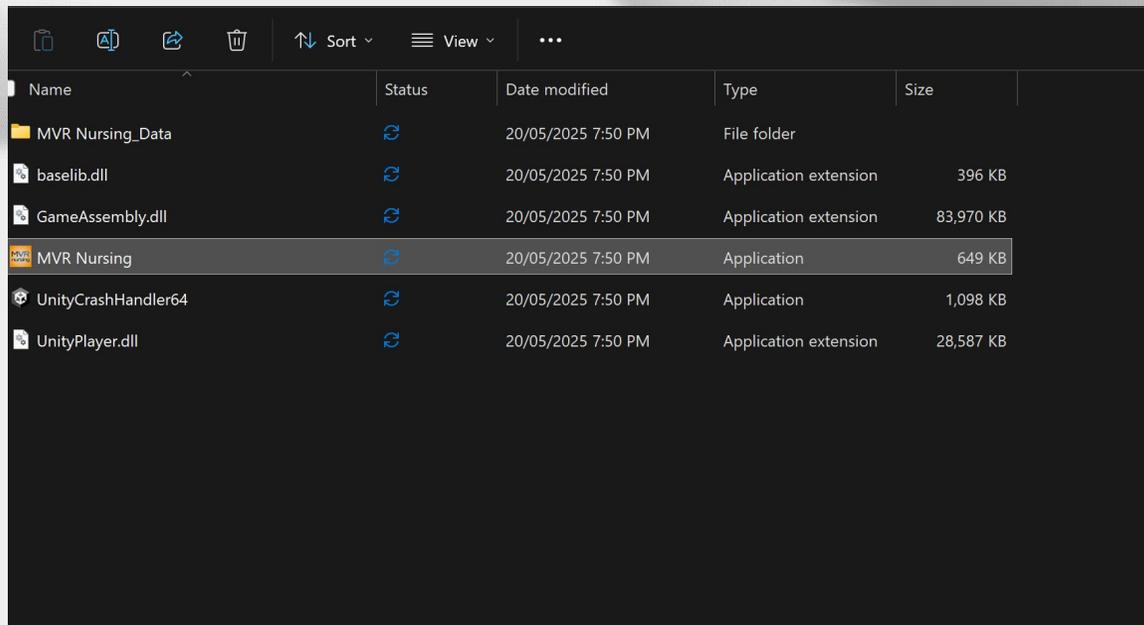

Name	Status	Date modified	Type	Size
MVR Nursing_Data		20/05/2025 7:50 PM	File folder	
baselib.dll		20/05/2025 7:50 PM	Application extension	396 KB
GameAssembly.dll		20/05/2025 7:50 PM	Application extension	83,970 KB
MVR Nursing		20/05/2025 7:50 PM	Application	649 KB
UnityCrashHandler64		20/05/2025 7:50 PM	Application	1,098 KB
UnityPlayer.dll		20/05/2025 7:50 PM	Application extension	28,587 KB

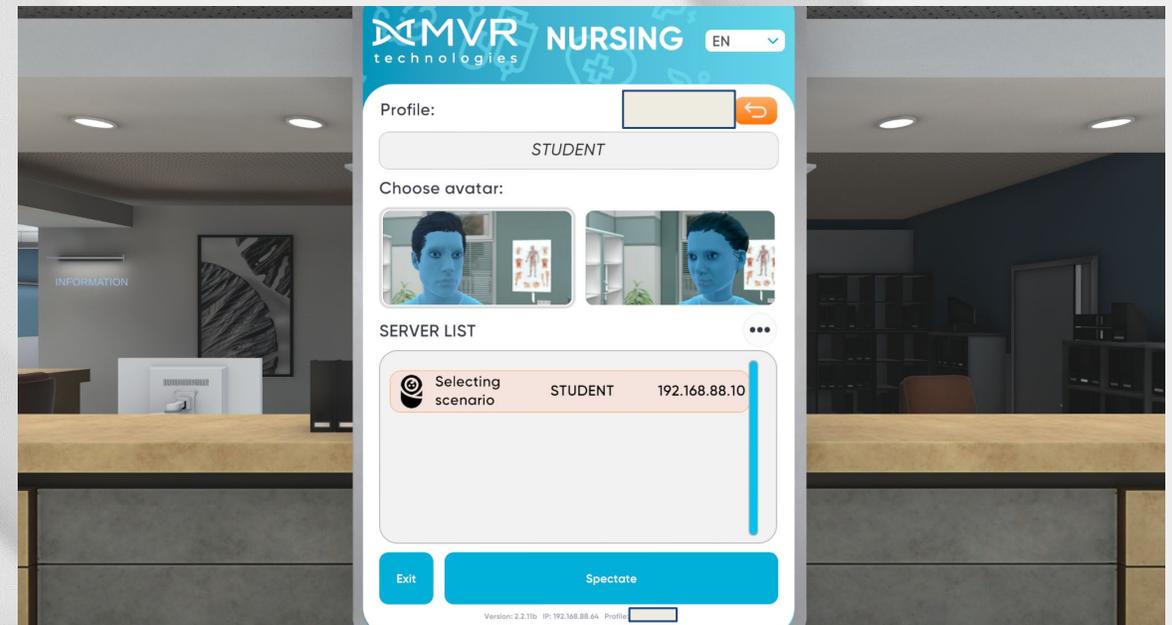

Be
READY

Technical support
service@mvredtech.com

Nasco
HEALTHCARE