



Be READY VR-Nursing Meta Quest 2 setup manual

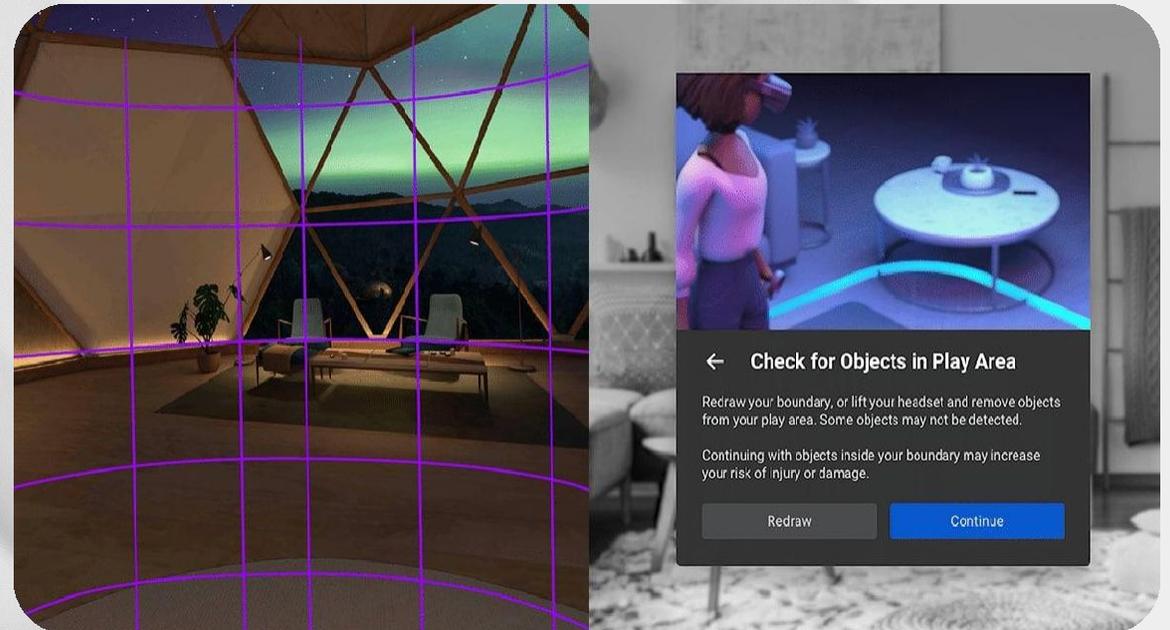


Set up the headset

- Charge the headset using a cable
- Hold the power button to turn on VR headset
- Connect to WI-FI network and install system updates
- Create the boundary or choose the stationary one
- Make sure the boundary zone is free of objects

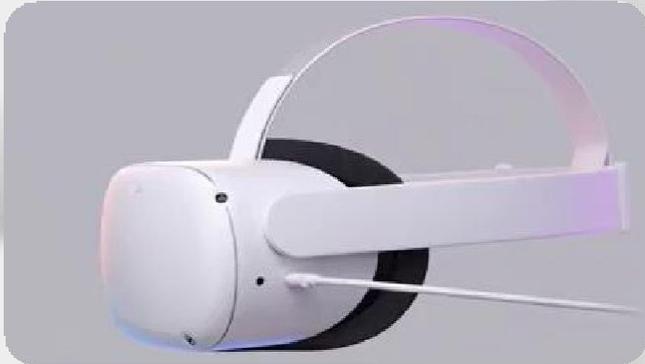




Set up the Meta account

- Create an account on auth.meta.com or use the existing one
- Send the email that you use for Meta account to the service@mvredtech.com
- Tech support team will provide access to the Be READY VR-Nursing application for your Meta account

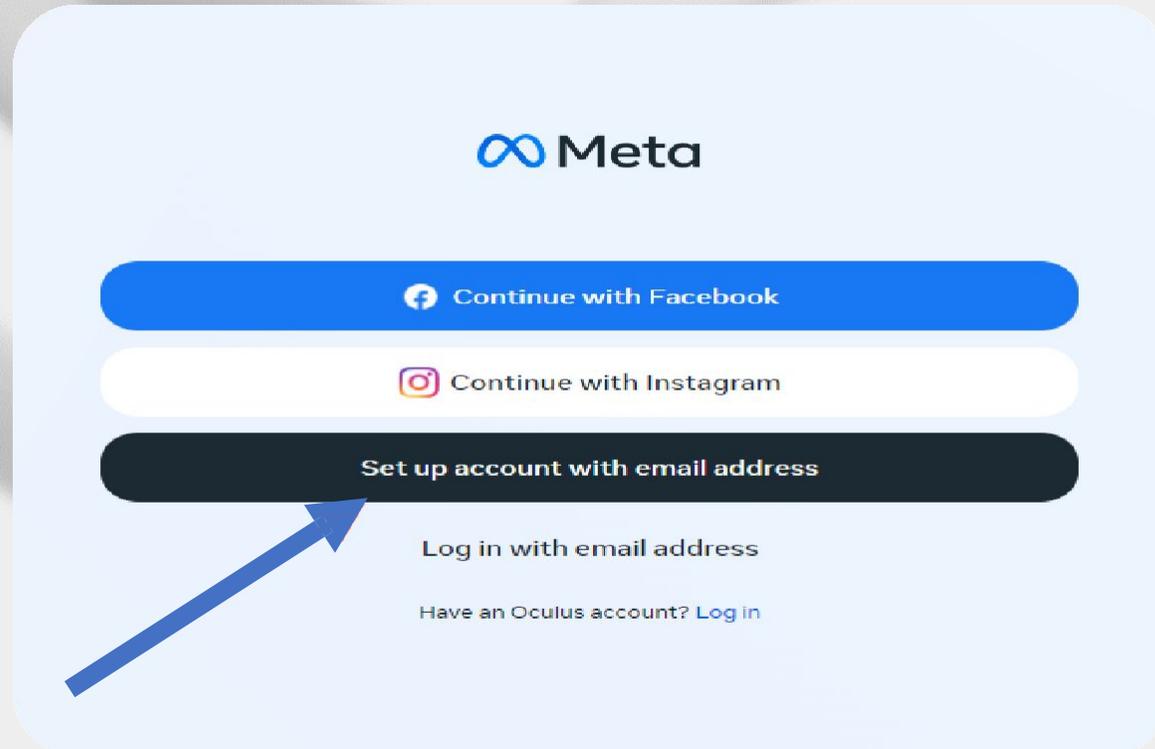

Pair the headset with Meta account

- Turn on the geolocation option on your phone
- Download the Meta Quest app
- Log in the Meta Quest app using email address
- Go to Menu -> Devices -> Pair new headset -> Quest 2
- Enter the code from the headset into the mobile application to finish the pairing

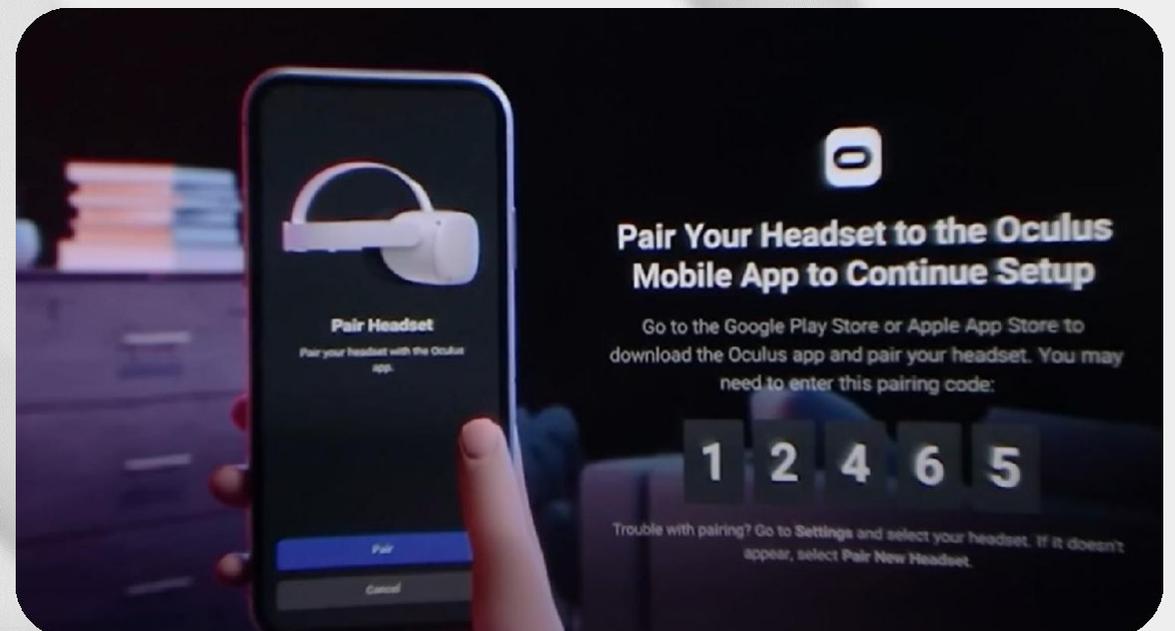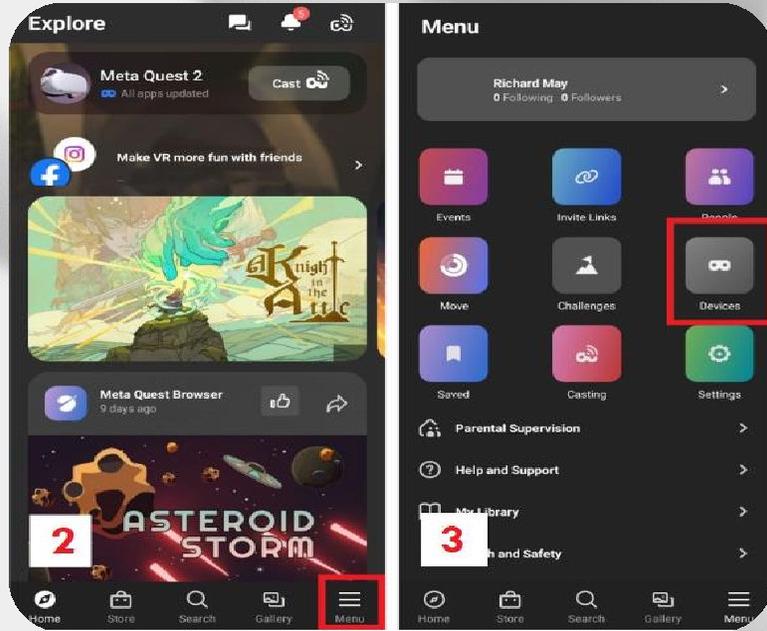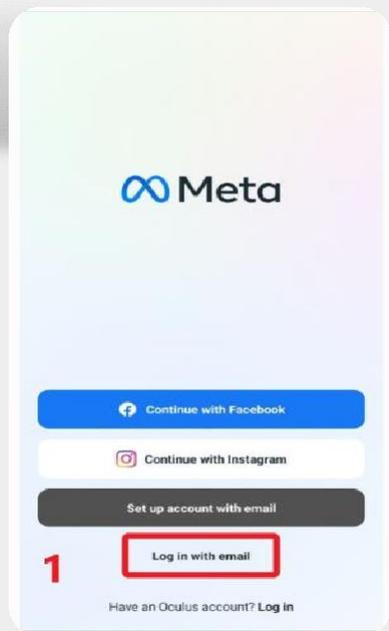



Install the application

- Go to the App Library and install the Be READY VR-Nursing application
- Launch the installed application from the App Library



Log in the Be READY VR-Nursing Mobile

- The Profile key is provided in the welcome letter to your e-mail
- Enter the Profile key on the start screen of the VR-Nursing and click “Confirm”

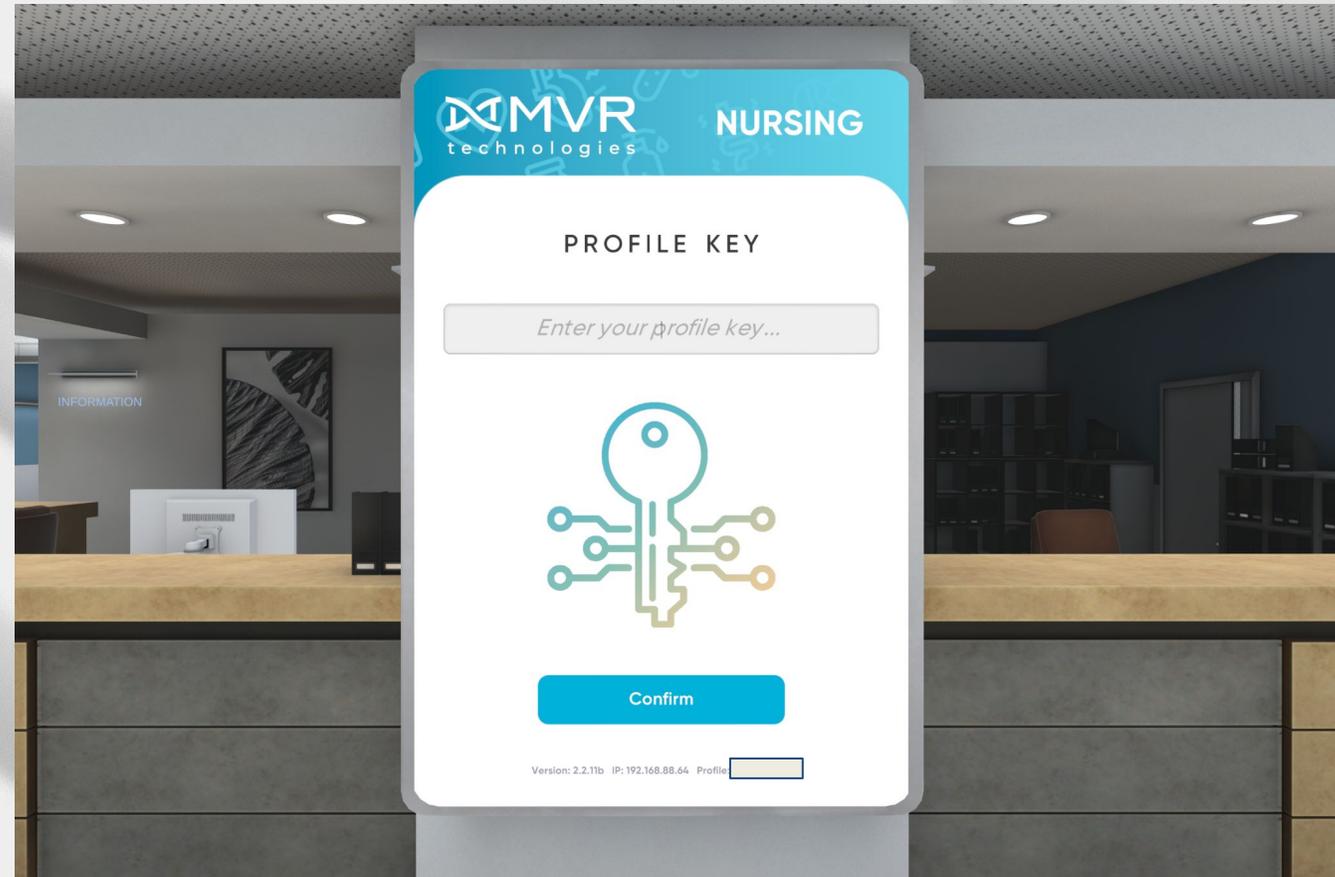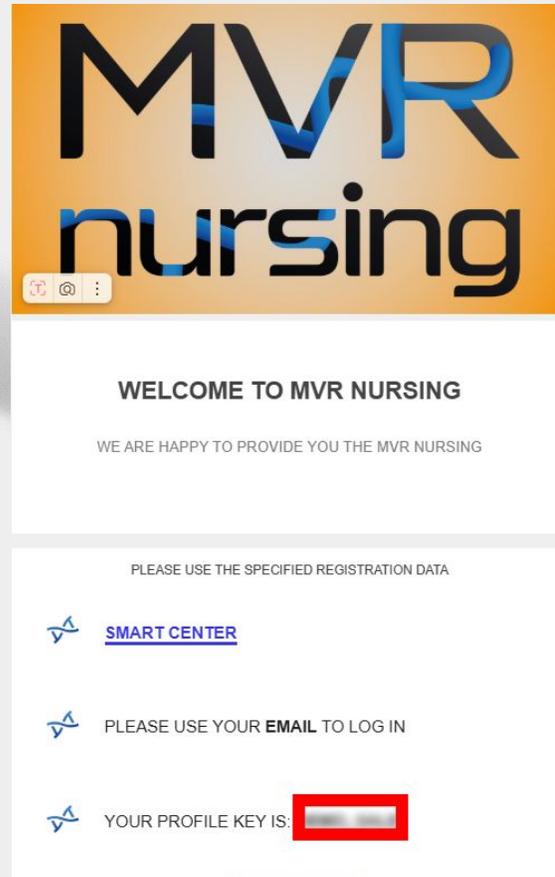

Log in the Be READY VR-Nursing Mobile

- Allow the application the access to file system
- The license GUID should be confirmed in your Smart Center

Log in Smart Center

- Go to [Smart Center](#)
- Login using email and password provided in the welcome letter and click “Sign in”
- Go to License activation tab in Smart Center
- Pick the product Nursing

Email

Password

Show password

Remember login details

[Sign in](#) [Recover password](#)

The screenshot shows the 'SMART CENTER' interface. On the left is a 'Filter' sidebar with sections for Calendar, Name, Group, Simulator, Clinical case, and Other. The main area displays a table of examination results. The table has columns: Date, Name, Group, Clinical case, Examination, License, Platform, Score, and Open. A blue arrow points from the 'Statistics' tab on the right to the 'Score' column of the table.

Date	Name	Group	Clinical case	Examination	License	Platform	Score	Open
X 2023-04-20 17:18:54	Twain Mark DemoSale		Acute appendicitis, uncomplicated	No	Yes	🖥️	5	Open
X 2023-04-19 17:41:02	Twain Mark DemoSale		Acute appendicitis, uncomplicated	No	Yes	🖥️	25	Open
X 2023-04-19 17:28:16			Acute appendicitis, uncomplicated	No	No	📱	40	Open
X 2023-04-19 16:27:31	Twain Mark DemoSale		Community-acquired non-serious segmental pneumonia S7 of the left lung Respiratory failure 3rd	No	Yes	🖥️	0	Open
X 2023-04-19 15:27:55	Twain Mark DemoSale		Acute appendicitis, uncomplicated	No	Yes	🖥️	0	Open
X 2023-04-19 14:19:38	Twain Mark DemoSale		Aortic insufficiency, Arterial hypertension, grade 2 Chronic heart failure, functional class 2	No	Yes	🖥️	0	Open
X 2023-04-19 14:17:46	Twain Mark DemoSale		Aortic stenosis, Chronic heart failure functional class 2	No	Yes	🖥️	3	Open
X 2023-04-19 14:16:09	Twain Mark DemoSale		Bronchial asthma persistent, moderate severity exacerbation, uncontrolled	No	Yes	🖥️	0	Open
X 2023-04-10 22:35:24	Twain Mark DemoSale		Community-acquired non-serious segmental pneumonia S7 of the left lung Respiratory failure 3rd	No	Yes	🖥️	100	Open
X 2023-01-13 19:10:44	Twain Mark DemoSale		Gastroesophageal reflux disease, erosive esophagitis of stage A	No	Yes	🖥️	3	Open
X 2023-04-13 19:09:13	Twain Mark DemoSale		Community-acquired non-serious segmental pneumonia S7 of the left lung	No	Yes	🖥️	0	Open

Confirm the Mobile license in Smart Center

- Your headset GUID is waiting for confirmation in the list of Mobile licenses
- Click the “tick” icon to confirm the device
- Headset GUID will appear in the list of Mobile licenses

Mobile license (VR device):

3 ID (VR device). 

Confirmation required.

1	100-010	201-2a2	<input checked="" type="checkbox"/> 
---	---------	---------	---



Mobile license (VR device):

3 ID (VR device). 


1	100-010		
2	<u>201-2a2</u>		

Possible error

- If you skip the Smart Center confirmation step the notification will appear
- Confirm the license in Smart Center and proceed with next steps


Finish VR-Nursing Mobile setup

- Reopen the application in the headset
- Enter your Profile key in the headset once again
- Download available cases from the Case Library

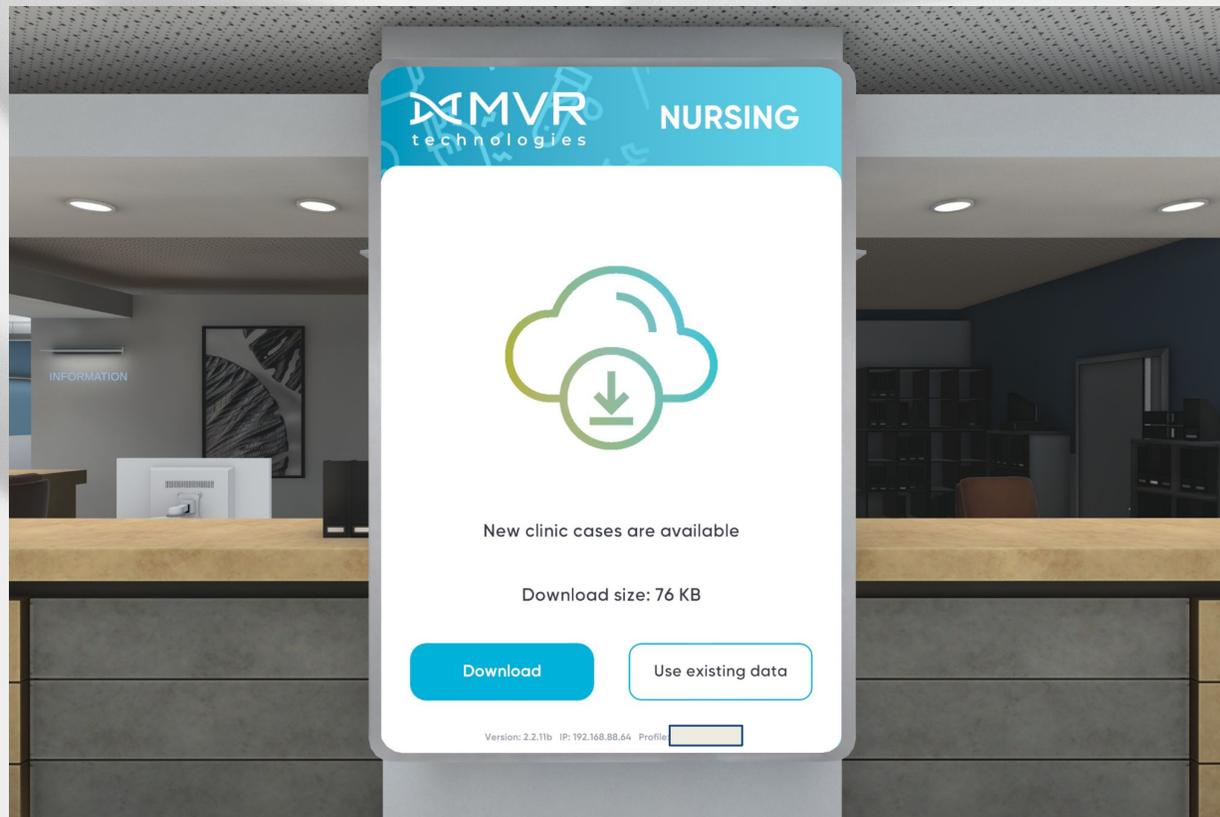

Finish the VR-Nursing Mobile setup

- Do not take your headset off while cases are downloading
- As soon as download is complete the VR-Nursing simulation will begin

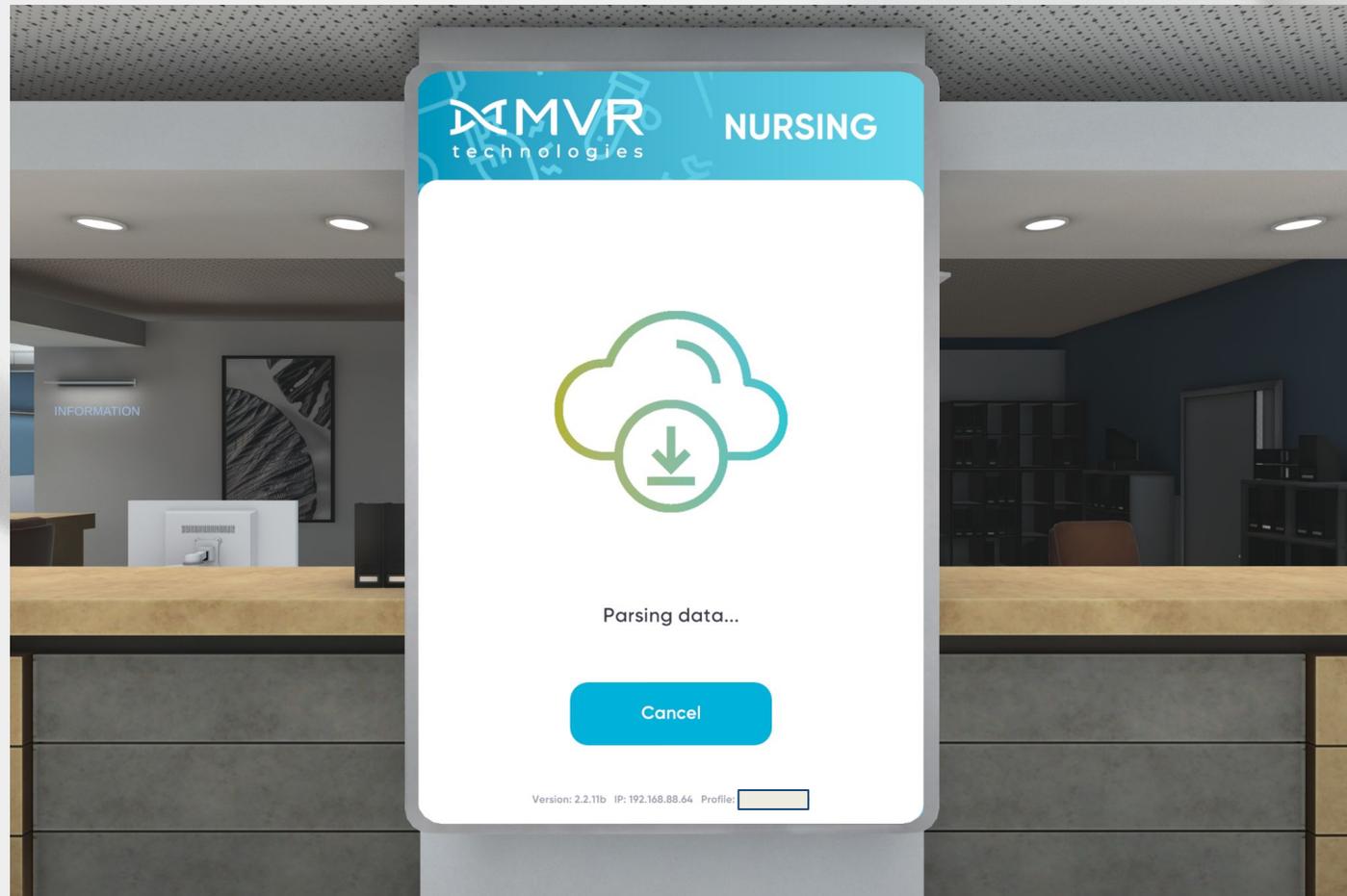

Useful links

Oculus Quest 2 setup:

<https://www.meta.com/en-gb/help/quest/articles/getting-started/getting-started-with-quest-2/set-up-meta-quest-2/>

Meta Quest application setup:

<https://www.meta.com/en-gb/help/quest/articles/getting-started/getting-started-with-quest-2/install-meta-quest-mobile-app/>

Be
READY

Technical support
service@mvredtech.com

Nasco
HEALTHCARE